NUBIS[®] SPORT

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NUBIS®

CONTENT

1. Introduction	3
2. Product overview	3
3. Assembly/ installation	4
4. Disassembly	5
5. Operation	5
6. Safety instructions	6
7. Cleaning and maintenance	7
8. Technical data	7
9. Accessories and Spare parts	8
10. Warranty and Liability	8

1. INTRODUCTION

Thank you for choosing NubisSport portable massage mattress.




NubisSport is a portable massage mattress, adapted to the conditions of massage in the field and created for fast recovery of the person treated. Suitable for outdoor use and for all types of terrain, it is extremely light and easily portable, even in an inflated state. It is very stable and ready for firmest massage techniques. It's easy to set up, when not in use it can be transported in a lightweight backpack.

In order to use NubisSport correctly and safely, it is important to read these User manual carefully before starting to use the product.

2. PRODUCT OVERVIEW

NubisSport consists of:

1. Massage Mattress
2. Head rest with breathing hole
3. Knee mat
4. Electric pump
5. Backpack.

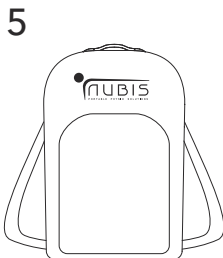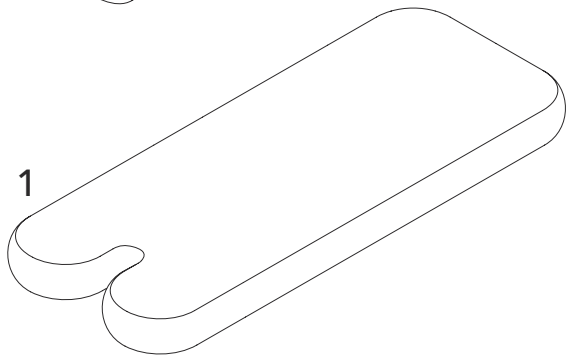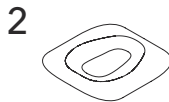
3. ASSEMBLY/ INSTALLATION

3.1. Take the NubisSport outside the cardboard packaging and check everything for possible transport damage.

3.2. Read this manual carefully before using the equipment.

3.3. Open the backpack and take out in following order:

1. Head rest
2. Electric pump
3. Mattress
4. Knee mat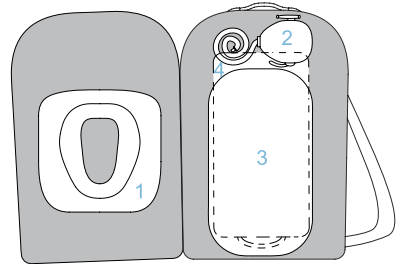


3.4. Mattress inflation:

1. Unwrap the mattress.
2. Unscrew the valve cap, push button into the "OUT" position.
3. Take the electric pump.
4.  Prior to connection to the mains supply, check that the voltage corresponds with that written on the plate.
5. Fit the black adaptor of the tube on the inflation side of the inflator (OUT).
6. Set the pump nozzle to the mattress valve.
7. Check that the ON/OFF switch is set to (0).
8. Plug the cable into the mains supply and turn the switch to the (I) position, the inflation will start. It takes approx. 35 seconds to inflate.
9. When the maximum pressure is reached, switch off the inflator and take out the tube.
10. After inflation tighten the valve cap (clockwise). Only the valve's cap can ensure final air tightness.

3.5. Head rest attachment and knee mat usage:

1. Attach the head rest using Velcro straps.
2. Put the knee mat on the floor, you can use it as a knee protection during therapy.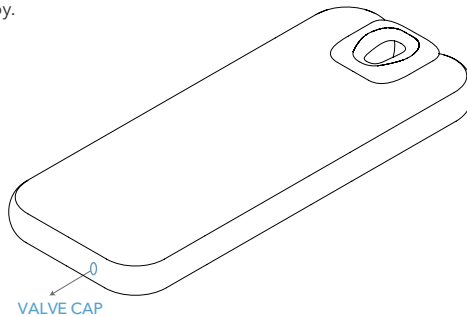


4. DISASSEMBLY

4.1. Remove head rest from the mattress.


4.2. Mattress deflation:

1. Unscrew the valve cap, push the button into the "IN" position.
2. Take the electric pump.
3. Fit the black adaptor on the deflation side of the inflator (IN).
4. Set the pump nozzle to the mattress valve.
5. Check that the ON/OFF switch is set to (0).
6. Plug the cable into the mains supply and turn the switch to the (I) position, the deflation will start.
7. When the mattress is completely deflated, switch off the inflator and take out the tube. Tighten back the valve cap.
8. Precisely fold the mattress and wrap it back into initial position.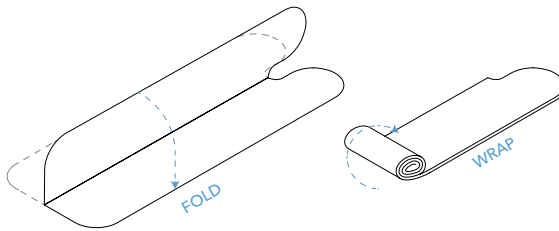

4.3. Insert into the backpack in following order:

1. Knee mat
2. Mattress
3. Electric pump
4. Head rest

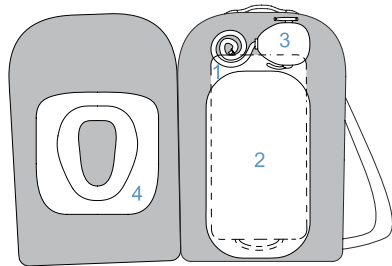5. OPERATION

5.1. Head rest adjustment:

Head rest is adjusted with Velcro straps and can be moved for 15 cm upwards, recommended for taller patients.

5.2. Air pressure mattress adjustment:

You can adjust the hardness of the mattress by adjusting the air pressure.

You can easily release the air pressure by pushing the button into the "IN" position.

You can increase back the air pressure according to the procedure in section 3. 4. (Mattress inflation).

6. SAFETY INSTRUCTIONS

6.1. General instructions:

- The product is meant to be used by competent personnel in massage field.
- Keep children and unqualified people away from equipment when no supervision is at hand.
- Use a clean mattress cover for every individual, in order to guarantee a hygienic use.

6.2. Mattress handling:

- Do not expose inflated mattress to extreme temperature conditions (allowed limit from -20°C to +50°C).
- Prevent damage to the upholstery from sharp objects.
- Do not fold and store the mattress, when wet. After cleaning or disinfecting, to prevent damages, allow material to dry properly before usage or storage.
- Follow carefully the cleaning and maintenance instructions of upholstery (Section 7 .), in order to prevent damage of the upholstery.

6.3. Inflation/ deflation process:

- Prior to connection of electric pump to mains supply, check that the voltage stated on the plate correspond with the available mains supply.
- Do not use the electric pump in humid environments or close to water, also never touch the power cable with wet hands (risk of electric shock).
- After inflating/ deflating the mattress, switch off the inflator immediately to avoid overheating the motor.
- Do not use the inflator with adapters others than the one supplied with the product as they could damage the device.
- Do not inflate beyond max. air pressure, stated in technical data. This cannot happen, when using original electric pump, supplied by Airfun d.o.o.

7. CLEANING AND MAINTANANCE

7.1. Cleaning the upholstery:

- Clean the upholstery with a damp cloth (use lukewarm water and a microfiber cloth).
- Remove stubborn marks with a pH-neutral soap.
- Never use agents that contain aggressive cleaners (strong detergents, solvents, bleaching or scouring agents).

7.2. Disinfecting the upholstery:

- Clean the upholstery first as described under point 7.1. before disinfecting.
- Follow the disinfectant's instructions for use.
- You can use disinfectants with light alcohol concentration or use Nubis disinfectant, recommended from Airfun d.o.o.

7.3. User maintenance:

- Regular cleaning and disinfecting of the upholstery.
- Anyway we recommend to regularly monitor main functions of the product and in case of any malfunctions contact authorized distributor or manufacturer.

8. TECHNICAL DATA

Product		Nubis SPORT
Code		100-7520
Mattress	Dimensions (L x W x Thickness)	190 x 75 x 20 cm
	Weight	4,9 kg
	Base/ cover material	DWF / PU coating
	Head section adjustment	0 - 15 cm (in length)
	Weight capacity	350 kg
Pump	Power supply	230V~50Hz
	Max power	1000W
	Max pressure	250/3.6 mbar/psi
	Insulation safety class	Class II (double insulated) 
Backpack	Dimensions	60 x 35 x 20 cm
	Transport weight (full backpack)	8,2 kg

Airfun d.o.o. reserves the right to change technical specifications.

9. ACCESSORIES AND SPARE PARTS

- 9.1. For the information on Nub is accessories, we refer to our catalogue or to our web site:
www.nubis-physio.com.
- 9.2. If spare parts or any other help is needed, please contact authorized distributor or Airfun d.o.o. for further assistance.

10. WARRANTY AND LIABILITY

10.1. Warranty:

- The warranty period for NubisSport is 1 year. The warranty starts with the day of Purchase.
- Should any repairs be necessary during warranty period, the customer must inform Airfun d.o.o. or your local dealer. The warranty must be verified by purchase invoice.
- The warranty obligates Airfun d.o.o. to carry the costs of repair of components which might fail during normal use inside the warranty period.
- Warranty is not applicable for defects or poor performance in following cases:
 - In the event of customer's improper use or neglect of NubisSport;
 - If customer doesn't follow requirements and recommendations of this User manual;
 - If any modifications or repairs are carried out by non-qualified service personnel, unauthorized by the manufacturer.

10.2. Product life expectance and repairs:

- This product's expected life is estimated to be 5 years.
- Repair and spare parts are available during life expectance, so 5 years from the date of purchase.
- Any repairs must always be performed by Airfun d.o.o. or any of its authorized technicians from local dealers.

10.3. Product liability:

The manufacturer or its suppliers or resellers cannot be liable for any consequences incurred by the user, the patient or the product itself as a result of improper use of the product, wrong interpretation or non-observance of this user manual, repairs carried out by unauthorized personnel.

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